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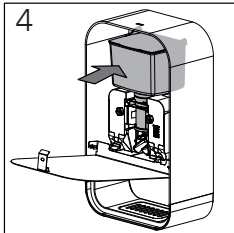
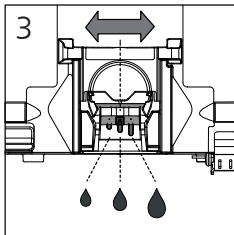
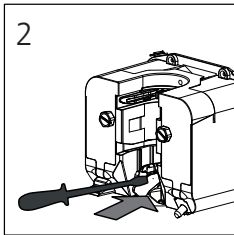
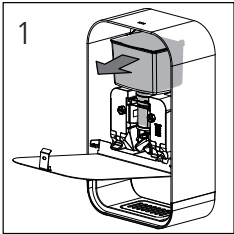
max. Dosiereinstellung 1,2 ml/Hub

E 9. Adjust dosing

max. dosage 1.2 ml/stroke

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E 10. Maintenance and cleaning instructions

This device is largely maintenance free. Regular emptying/cleaning of the drip tray/drip grid and wipe disinfection/cleaning of the outer surfaces of the dispenser are necessary. The cleaning of the base part including the drip tray can be done with door opened (see point 4: Opening the device / position 3). Soap and disinfectant residues inside the device must be removed at regular intervals.

NOTE Cleaning agents must be compatible with plastics. **Do not disassemble the unit. Use only original spare parts from the manufacturer.**

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E 11. Disposal of used electrical and electronic devices

The symbol on the product or its packaging indicates that this product should not be treated as standard household waste. Device must be taken to a recycling point for electrical and electronic devices. For more information, contact your local municipality, municipal waste disposal service, or the company from which you purchased this product.

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E 12. Statutory information on battery disposal

Batteries should not be disposed of as household waste. As a consumer, you are required by law to return used batteries. Old batteries should be surrendered to the public collection points in your municipality or to anywhere where batteries of this type are sold.

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D 13. Troubleshooting

- Do the batteries need to be replaced?
- Is the polarity of the batteries correct?
- Are the pump and container inserted correctly?
- Is the correct filling material used? (Please pay attention to the label on the container)

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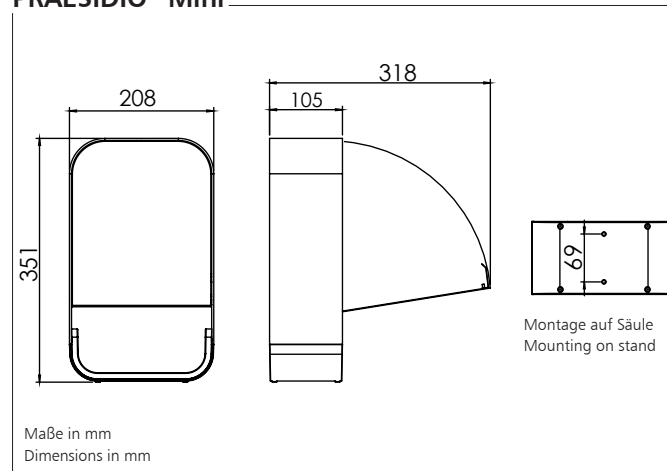
D 14. Technical data

Housing: aluminium, matte silver anodized
Door: stainless steel, brushed
Base: ABS-Plastic
Drip tray: stainless steel
Key: ABS-Plastic
Lock: PRAESIDIO Mini spring lock (DIN 1.4310)
Container/Pump: PCR-Plastic, removable, 600 ml
Dosage: max. 1,2ml/stroke
Batteries: 4x Alkaline D batteries, 1,5 V, exchangeable, non-rechargeable
Total weight (with batteries): 2,9 kg

Conformity: RoHS compliant
Machinery directive 2006 / 42 / EC
EMV- directive 2004 / 108 / EC

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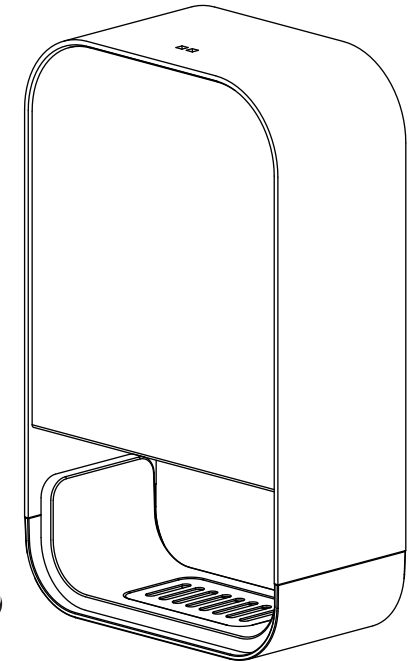
The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to guarantee fault-free functionality. OPHARDT Hygiene assumes no liability for any damages or costs which arise as a result of incompatibility of filling material and dispenser systems.

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E 1. Scope of delivery

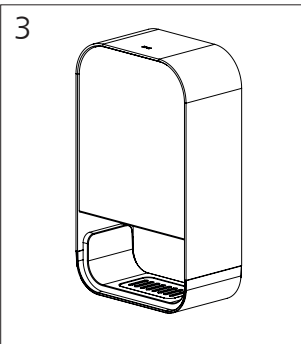
- Dispenser
- Key
- Instruction manual
- Batteries (4 x Type D)

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E 2. Intended use

Automatic dispenser for alcohol-based hand disinfectant and others depending on the pump version. Refill volume max. 600 ml. Only for use in enclosed rooms without risk of explosion in non-condensed environment.

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Wandhalter: Art.-Nr. 1424814
Säule: Art.-Nr. 1424837

E 3. Mounting

The device is intended for table mounting. Make sure the device is standing stable. Also suitable for wall or stand mounting. Wall bracket: Art. no. 1424814
Stand: Art. no. 1424837

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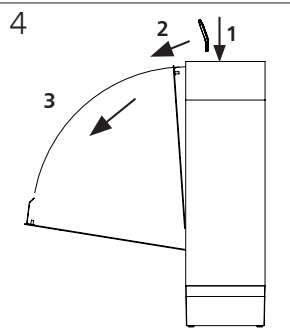
NOTE If free standing please ensure that:

- The surface is level and secure
- It is clear of any passing or moving objects.
- If the dispenser is knocked any spillage should not affect nearby equipment.
- Check critically whether your application allows free-standing operation. If necessary, use the wall bracket or the column for fixation.

VORSICHT

- Do not place or install in traffic routes and direct work areas.
- Select installation / mounting location so that it prevents access by small children.
- Children must be instructed and supervised during use.

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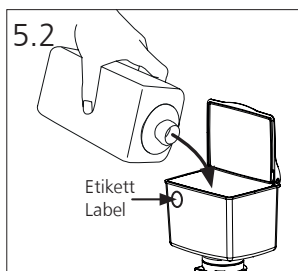
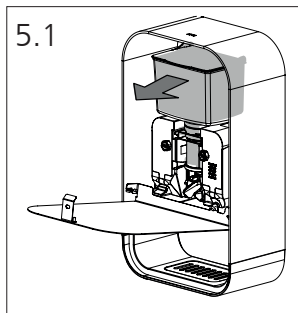
E 4. Opening the device

To open the device, insert the key into the dispenser from above and press the spring tongue down (1). The door unlocks and opens a gap to the front (2). Then swing the door down as far as it will go. The safety switch prevents injuries and operating errors due to unintentional triggering of the dispenser when the bonnet is open.

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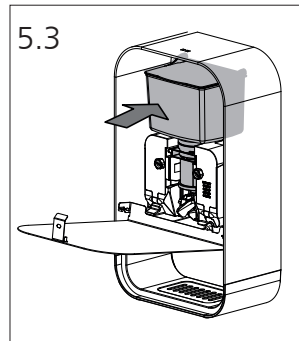
E 5. Filling/replacing/ installing the container with pump

Filling the container: Open the device. Remove the container (Fig. 5.1) Lift the lid of the container upwards to the detent position and fill the filling material into the container (Fig. 5.2).

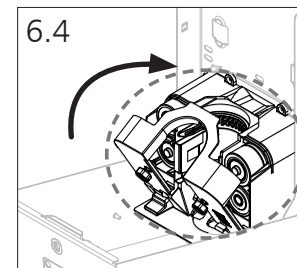
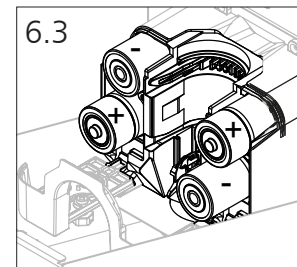
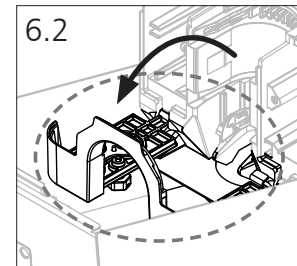
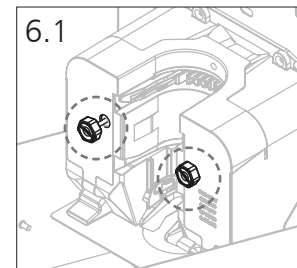
NOTE

Only use suitable filling material in accordance with the pump type (liquid soap / foam soap / alcohol-based disinfectant). See label on the container. Only use free-flowing products agents without abrasives. Tests are required before new filling materials are used for the first time.

Installing/Replacing the container with pump: Open the device. To remove the pump, pull out the container with pump to the front. When replacing the pump and container, ensure the pump is correctly seated in the guide rails. (see fig. 5.3)



NOTE

Container and pump can be replaced together so that other filling materials can be used.

E 6. Insert/change batteries

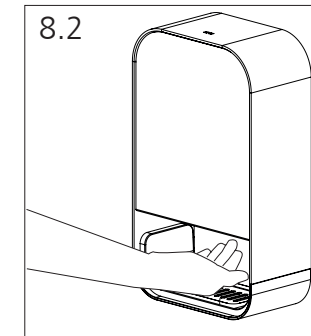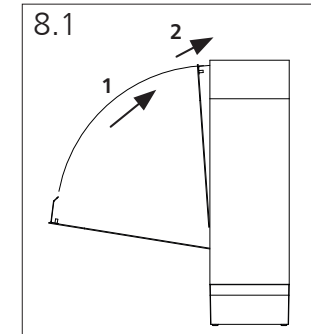Open the device. For initial start-up, remove the contact isolating strip completely. The device is immediately ready for operation. To change the batteries, loosen the screws of the battery cover (Fig. 6.1) and lift it forward (Fig. 6.2). Remove used batteries and insert new batteries into the compartments provided, according to the correct polarity (Fig. 6.3). Then fold the cover back up (Fig. 6.4) and tighten it with both screws. Replace the container (Fig. 7). Use only batteries of the mono cell type (type D) with 1.5V voltage. Do not insert rechargeable batteries.

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E 8. Closing the device/Operating

To close the device, raise the door in order to engage it in the end position in the catch. (see fig.8.1) Operation: Hold your hand under the outlet (see fig. 8.2).

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